ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE: The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user or caregiver should be able to read, understand and be physically capable of performing all the directions, warnings and cautions provided in the information for use.

INDICATIONS: The Procare Maternity Belt is designed to provide support to the lumbar spine and abdominal area during pregnancy. It may be suitable for the symptomatic relief of lumbar pain and discomfort due to stretching and expansion of the abdomen. Overall, it provides comfort and support to the lower back and abdomen.

PERFORMANCE CHARACTERISTICS: Soft-good/semi-rigid designed to restrict motion through elastic or semi-rigid construction.

CONTRAINDICATIONS: If you have skin disorders/injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional.

WARNINGS AND PRECAUTIONS:

- As this belt provides abdominal support, please consult your physician if using post-operatively as there is an increased risk of blood clots.
- For maximum efficacy, ensure the correct size of support is selected and the side pulls are adjusted to the required fit.
- Do not use over open wounds.
- If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.
- Do not use this device if it was damaged and/or package has been opened.

NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

A) Place the abdominal support pad in a lifting position under the abdomen. Fasten in place by wrapping the narrow band of elastic around the body and pressing the contact closure hook to the abdominal support pad.

B) With abdominal support pad in place, wrap belt around the body without stretching. Hook the ends to the abdominal support pad.

C) Stretch upper abdominal strap and hook.

D) With the upper abdominal strap in place, grasp both ends of the belt and unhook them from the abdominal support pad. Now stretch them as far as you can and rehook.

Important: After following instructions for above, have patient lie on back and detach contact closure. Then stretch belt as far as possible and reattach.

E) As the baby grows, the Maternity Belt expands accordingly. The belt accommodates the growing abdomen simply by fastening farther away from the center of the abdominal support pad.

USE AND CARE: Hand wash in cold water with mild detergent. Rinse thoroughly and air dry. Do not bleach.

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

For single patient use only.
Not made with natural rubber latex.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, NO EXISTING GUARANTEE ALONE WILL BE PROVIDED THROUGH THE USE OF THIS PRODUCT.

CAUTION: FEDERAL LAW (U.S.A.) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PROFESSIONAL.

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