## ITALIANO

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13-1416-0-00000 REV C - 2021-01-22

## ENGLISH

**BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.**

**INTENDED USER PROFILE:** The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

**INDICATIONS:** The ProCare Calf Sleeve is designed to support the calf. It may be suitable for use following a soft tissue injury. Soft-good/semi-rigid designed to restrict motion through elastic or semi-rigid construction.

**WARNING:** Neoprene supports should not be worn by individuals with known susceptibility to dermatitis or allergies to neoprene by products.

### WARNINGS AND PRECAUTIONS:

- It is recommended to have the device fitted by a suitably qualified healthcare professional.
  - If you develop an allergic reaction and/or experience itchy, red skin after coming into contact with any part of this device, please stop using it and contact your healthcare professional immediately.
  - If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
  - Do not use over open wounds.
  - Do not use this device if it was damaged and/or packaging has been opened.
- NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

### APPLICATION INFORMATION:

**A** Roll a small cuff in the larger opening of the support.

**B** Grasp the cuff and pull the support onto the leg so that the affected area is covered.

**USE AND CARE:** Hand wash in cold water with mild soap, rinse thoroughly. AIR DRY. Note: If not rinsed thoroughly, residual soap may cause skin irritation or deteriorate material.

**WARRANTY:** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

**FOR SINGLE PATIENT USE ONLY.**

**NOT MADE WITH NATURAL RUBBER LATEX.**

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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©2021 DJO, LLC