## ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

### INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

**INTENSE USE/INDICATIONS:** Treatment of wrist sprains, strains, and Carpal Tunnel Syndrome (CTS).

**CONTRAINDICATIONS:** None

### WARNINGS AND PRECAUTIONS:

- If you have skin disorders/injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional.
  - If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
  - The support should be snug but not impair circulation.
  - Do not use over open wounds.
  - Do not use this device if it is damaged and/or the packaging has been opened.
- NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

### APPLICATION INFORMATION:

- 1) Loosen all straps and position hand in brace. Make sure the palmer stay is positioned comfortably below the palmer crease and aligned with forearm.
- 2) Position adjustable dorsal stay casing to ensure maximum support and proper fit. Stay casing may be moved proximal or distal for desired controlled range-of-motion.
- 3) Use single pull strap to secure body of brace. Straps should be snug but not constrict.
- 4) Secure strap between thumb and forefinger.

**CLEANING INSTRUCTIONS:** Hand wash in warm water using mild soap, rinse thoroughly. AIR DRY. Note: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

**WARRANTY:** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

**PATENT PENDING. FOR SINGLE PATIENT USE ONLY.**

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DJO GmbH