## ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

**INTENDED USE/INDICATIONS:** Treatment of wrist sprains, strains, and Carpal Tunnel Syndrome (CTS).

**CONTRAINDICATIONS:** None

**WARNINGS AND PRECAUTIONS:** If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

### APPLICATION INFORMATION:

- Loosen all straps and position hand in brace. Make sure the palmer stay is positioned comfortably below the palmer crease and aligned with forearm.
- Position adjustable dorsal stay casing to ensure maximum support and proper fit. Stay casing may be moved proximal or distal for desired controlled range-of-motion.
- Use single pull strap to secure body of brace. Straps should be snug but not constrict.
- Secure strap between thumb and forefinger.

**CLEANING INSTRUCTIONS:** Hand wash in warm water using mild soap, rinse thoroughly. AIR DRY. Note: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

PATENT PENDING

FOR SINGLE PATIENT USE ONLY.

LATEX FREE

**WARRANTY:** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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© DJO, LLC 2009